Hydroactive Antibacterial Wound Gel

CICA-M[®] Gel

COMPOSITION

CICA-M soft gel contains,
Medical grade honey.....40%
Medical grade hypoallergenic lanolin(Medilan)
Vitamin C
Vitamin E

Category: Antibacterial

 **Dosage Form:** GEL

INDICATIONS

Superficial and acute wounds e.g. cut, abrasion and donor sites. Superficial and partial thickness burns (1st and 2nd degree). Chronic wounds e.g. pressure ulcer, venous, arterial and diabetic ulcer. Fungating wounds (to help deodorised and debride). Colonised acute wounds and (postoperative) surgical wounds.

When CICA-M is in contact with the wound, fluid is drawn from the surrounding tissues, this creates a moist wound healing environment. This stimulates the wound healing process by facilitating autolysis of necrotic and devitalized material, aids wound bed preparation and promotes epithelization.

CICA-M

- Has Antibacterial properties
- Dilutes gradually and can be removed easily as per local wound cleaning protocol.
- Prevents the secondary dressing adhering to the wound bed.
- Due to the use of natural ingredients, each may vary in color.

CONTRAINDICATIONS

No contraindications are known to date. Do not use on patients who are sensitive to the product or any of its components.

SPECIAL PRECAUTIONS

- 1) Always consult the doctor for wound management device.
- 2) Do not use new products that are damaged or expired.
- 3) Inspect and clean the wound according to local protocol.
- 4) Due to autolytic debridement the wound may appear to increase in size after the dressing changes. This is normal and occur as the devitalised tissue is removed, preparing the wound bed for healing.
- 5) In the event of intolerance to CICA-M or one of its components, remove it using appropriate technique and clean the area carefully.
- 6) CICA-M is for single patient use; the product may only be reuse with the same patient.
- 7) Always close the tube tightly after use.

- 8) After opening, use the product within three months.
- 9) Dispose of the product as per local protocol.

FREQUENCY OF USE

This depends upon the amount of exudate and the needs of the wound. Re-apply the product when it has dissolved (\pm once every 24-48 hours)

Instruction for Use:

Application to the Wound:

1. Use the sterile spatula to distribute a thin layer of CICA-M either directly to the wound area and the surrounding tissue or on to sterile gauze first.
2. Cover and secure with the appropriate secondary dressing depending on the amount of wound fluids.
3. Inspect the wound and surrounding tissue regularly, as per regular protocols.

A. Tube:

1. Remove the lid, clean the mouth of the tube and top of the lid with an alcohol swab.
2. Push the top of the lid into the foil seal that covers the nozzle, do not squeeze the tube at this time.

STORAGE INSTRUCTIONS

Store at room temperature between 15°C to 25°C. Avoid direct sunlight and moisture. CICA-M is natural product and it may become liquid under higher temperatures. If this expected to be problem place the product in a refrigerator prior to use.

GENERIC HEALTHCARE PVT. LTD.
29A, Shivaji Nagar, Pune-411005.
INDIA
export@ghpl.co
© Trade mark

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